

Sure-Vue® Signature Mono Test - Operator Competency Exam

It is recommended that operator competency in performing this test is documented following initial training. Consult local, state and federal regulations and/or your accreditation agency for additional information on training requirements.

Operator Name (printed): _____ **Employee Number:** _____

Unit, Clinic, or Department: _____ **Training:** _____
(Initial, Annual, Re-Training)

Practical Training Inservice:

- Procedural review, including control requirements
- Demonstration of the test procedure
- Successful performance of the **Sure-Vue® Signature Mono** test procedure
- Test interpretation and results

I have read and understood the complete Sure-Vue® Signature Mono test procedure, and have been trained in the above test and test procedure.

Operator Signature: _____ **Date:** _____

Competency Exam:

The following exam is administered as proof of competency for personnel performing the **Sure-Vue® Signature Mono** test using whole blood patient samples. Please circle your response to each question.

- Using the Sure-Vue® Signature Mono test, how many drop(s) of diluent should be added to the test tube?**
 - One drop of diluent
 - Three drops of diluent
- After filling a capillary tube with fingertip blood end to end, dispense how much of the blood into the test tube?**
 - 1 drop of blood
 - All of the blood
- In order for the test to be valid, a control line must be present.**
 - True
 - False
- Results from the Sure-Vue® Signature Mono test strip must be read at _____ minute(s).**
 - one
 - two
 - five
 - any point after addition of test strip

5. Interpret the following results:

#1



Device #1 is showing a(n) _____ result:

- A. Positive
- B. Negative
- C. Invalid

#2



Device #2 is showing a(n) _____ result:

- A. Positive
- B. Negative
- C. Invalid

#3



Device #3 is showing a(n) _____ result:

- A. Positive
- B. Negative
- C. Invalid

For Program Administrator Use Only!

Operator Score: _____ Operator Status: _____
(Passed or Additional Training Required)

If additional training required: Date scheduled: _____
Date completed: _____ Operator Status: _____

Program Administrator Signature: _____ Date: _____
(or designee)

Sure-Vue® Signature Mono Test - Operator Competency Exam - ANSWER KEY

Competency Exam Answers:

1. Using the Sure-Vue® Signature Mono test, how many drop(s) of diluent should be added to the test tube?
A. One drop of diluent
2. After filling a capillary tube with fingertip blood end to end, dispense how much of the blood into the test tube?
B. All of the blood
3. In order for the test to be valid, a control line must be present.
A. True
4. Results from the Sure-Vue® Signature Mono test strip must be read at _____minute(s).
B. five
5. Interpret the following results:

#1



Device #1 is showing a(n) _____result:

- A. Positive

#2



Device #2 is showing a(n) _____result:

- C. Invalid

#3



Device #3 is showing a(n) _____result:

- B. Negative