

Sure-Vue®

hCG-STAT Serum

Control Set

INTENDED USE

Sure-Vue® hCG-STAT Serum Controls are intended to be used to monitor the performance and accuracy of Sure-Vue® Serum/Urine hCG-STAT pregnancy test kits. These controls are for professional use only.

SUMMARY AND EXPLANATION

The use of known controls in the laboratory is invaluable. It is important to verify testing procedures to confirm the validity of the results reported. Testing with Sure-Vue® hCG-STAT Serum Controls will provide assurance that Sure-Vue® Serum/Urine hCG-STAT pregnancy test kits are performing properly.

REAGENTS AND MATERIALS

1. 10 mIU/mL hCG Positive Serum Control contains normal human serum and sodium azide 0.2%.
2. 250 mIU/mL hCG Positive Serum Control contains normal human serum and sodium azide 0.2%.
3. Negative Serum Control contains sodium azide 0.2%.

PRECAUTIONS

1. For *In Vitro* diagnostic use only.
2. Refer to the Sure-Vue® Serum/Urine hCG-STAT package insert for specific precautions.
3. Do not use controls if they are cloudy or if precipitates are observed in the vials. This may be an indication of reagent instability or deterioration.
4. These controls contain 0.2% of sodium azide which may react with lead and copper plumbing to form explosive metal azides. Drains should be flushed thoroughly with water after disposing of controls to prevent azide buildup.
5. Do not use controls beyond the

expiration date.

6. Materials manufactured from source plasma. Each donor unit of plasma used in the preparation of this material was tested and found negative for HbsAG, HIV ½, HIV-lag and HCV by FDA approved methods. Specimens and controls should be considered potentially hazardous and handled in the same manner as an infectious agent.

STORAGE

Sure-Vue® hCG-STAT Serum Controls are to be stored refrigerated (2° - 8°C) for the duration of the shelf-life. The controls must be equilibrated to room temperature (15° - 30°C) before use.

PROCEDURE

Directions for Use

1. Allow the controls to reach room temperature prior to testing. The controls are ready to use. No dilution is required.
2. The controls are used in place of the specimen and should be tested according to the Sure-Vue® Serum/Urine hCG-STAT package insert.

INTERPRETATION OF RESULTS

Positive Results

The 10 mIU/mL hCG and 250 mIU/mL hCG Positive Serum Controls should produce positive results.

Negative Results

The Negative Serum Control should produce a negative result.

Invalid Results

See the "Invalid Results" section from the Sure-Vue® Serum/Urine hCG-STAT package insert. The test should be repeated.

LIMITATIONS OF THE PROCEDURE

1. These controls are formulated for use as quality control specimens in the reagent

verification of Sure-Vue® Serum/Urine hCG-STAT pregnancy test kits. See the Sure-Vue® Serum/Urine hCG-STAT package insert for guidelines in procedure and interpretation.

2. Refer to the Sure-Vue® Serum/Urine hCG-STAT package insert for further limitations.

EXPECTED VALUES

The 10mIU/mL hCG and 250 mIU/mL hCG Positive Serum Controls should produce positive results. The Negative Serum Control should produce a negative result. If the desired result is not achieved, it may be an indication of the following:

1. The test kit is not performing properly.
2. The test was not performed correctly according to the package insert.

Invalid results should be analyzed to determine probable causes and to provide solutions for corrective actions.

PERFORMANCE CHARACTERISTICS

Sure-Vue® hCG-STAT Serum Controls have been designed to produce correct results in Sure-Vue® Serum/Urine hCG-STAT pregnancy test kits. These controls have been tested with Sure-Vue® Serum/Urine hCG-STAT pregnancy test kits and were found to produce satisfactory results.¹

REFERENCES

1. Data on file

For Technical Assistance: 1-888-727-3315

To Order:

Phone: 1-800-640-0640

Fax: 1-800-290-0290

www.fishersci.com

Revised 12-02

® Sure-Vue is a registered trademark of Fisher Scientific Company, L. L. C.