



# SUIT UP

## Medicom has you covered

Medicom protective apparel is made of high-quality materials and designed to provide complete coverage that complies with all applicable regulations. It provides the reliable protection you need without compromising comfort or style and helps create a reassuring environment for patients.

### Benefits of single-use protective apparel



Reduced risk of cross-contamination



Always clean, as packaging remains sealed until use



No laundering required

### Protective Apparel Best Practices

Protective apparel must be worn and disposed of correctly. We encourage you to share these guidelines with your staff:

#### Donning protective apparel

- Perform hand hygiene before removing item from package.
- Select appropriate garment for intended use (length, coverage, fluid resistance, etc.)
- Fasten correctly: tied at neck and waist to ensure comfortable fit, adequate coverage and freedom of movement.
- Protective apparel should not be worn in restrooms, personal offices or break rooms.

#### Doffing protective apparel

- Remove apparel immediately after use. Do not re-use.
- Perform proper hand hygiene after removing apparel to prevent cross-contamination.
- Turn garment inside out when removing to keep infectious material inside.
- Designate a safe receptacle in which to dispose of protective apparel after use.

#### When to change apparel

Change protective apparel between patients and as soon as possible when penetrated by blood or other potentially infectious fluids.

# SafeWear™

Premium Protective Apparel



## Discover the complete range

Available in Hipster Jackets and High Performance Lab Coats, made from top-quality materials and designed to provide reliable barrier protection without sacrificing comfort or style.

### Hipster Jacket™

Premium hip length jacket

	XS	S	M	L	XL	2XL	3XL
Deep Blue*		19910813	19910814	19910815	19910816	19910817	19910818
Pretty Pink		19910819	19910820	19910821	19910822		
Plum Purple*	19910846	19910823	19910824	19910825	19910826		
Soft Blue*		19910827	19910828	19910829	19910930		
White Frost*		19910831	19910832	19910761	19910762		
Tropical Teal		19910763	19910764	19910765	19910766		
Poppy Pink		19910767	19910768	19910769	19910770		
Jet Black		19910771	19910772	19910773	19910774		

12 units/package. 5 packages/case.

### High Performance Lab Coat™

Premium lab coat

	XS	S	M	L	XL	2XL	3XL
Deep Blue*		19910784	19910785	19910786	19910787	19910788	19910789
Pretty Pink		19910790	19910791	19910792	19910775		
Plum Purple*		19910776	19910777	19910778	19910779		
White Frost*		19910780	19910781	19910782	19910783		
Soft Blue		19910833	19910834	19910835	19910836		
Tropical Teal		19910841	19910842	19910843	19910844		
Jet Black		19910837	19910838	19910839	19910840		

12 units/package. 5 packages/case.

#### FIT

- Thoughtful designs offer ease of movement, as well as secure closure and easy removal.
- Special features such as front snaps, back slits and neck and waist ties, provide a safe, comfortable and personalized fit.
- Lightweight, soft, supple fabrics keep wearers cool and comfortable.
- SafeWear protective apparel is latex free.
- Available in a wide range of sizes, from XS - 3XL.

#### FUNCTION

- SafeWear protective apparel complies with recommended OSHA best practices.
- Available in a wide range of material weights and polypropylene/SMS variants to meet a variety of needs while maintaining breathability.
- Premium materials provide superior protection, reliability and durability.
- Hipster Jackets and Lab Coats are made from AAMI PB70 Level 2 tested material.

# SafeBasics™

SafeBasics protective apparel provides a basic level of protection at an attractive price.

## Isolation Gowns

Single-use gown for procedures requiring protection from spray and splatter. Fluid-repellent design provides added protection against cross-contamination.



Catalog No.	Description	Unit/Pkg	Pkg/Case
296368443	Isolation Gown Reg - Yellow	10	5
19910742	Isolation Gown XL - Yellow	10	5
19910553	Isolation Gown Reg - Blue	10	5

## Lab Gowns

Front snap closure, knitted cuffs and collar for comfortable, easy on-and-off fit.



Catalog No.	Description	Unit/Pkg	Pkg/Case
19910554	Lab Gown S - Sky Blue	10	5
19910555	Lab Gown M - Sky Blue	10	5
19910556	Lab Gown L - Sky Blue	10	5
19910557	Lab Gown XL - Sky Blue	10	5

## Fluid Impervious Gowns

Polyethylene coating provides fluid resistance for procedures involving the potential for cross-contamination.



Catalog No.	Description	Unit/Pkg	Pkg/Case
19910744	Fluid Impervious Gown Reg - White	10	5
19910745	Fluid Impervious Gown XL - White	10	5

## Bouffant Caps

Hair and head fluid protection with breathable, low-linting design.

## Beard Covers

Face protection with an elastic edge and headband loop for optimal fit and comfort.



Catalog No.	Description	Unit/Pkg	Pkg/Case
19910549	Bouffant Caps 21" White	100	5
19910548	Bouffant Caps 21" Blue	100	5
19910549	Bouffant Caps 24" White	100	5
19910550	Bouffant Caps 24" Blue	100	5
19910803	Beard Covers, White	100	10

## Shoe Covers

Footwear protection with fluid repellent, slip-on covers.



Catalog No.	Description	Unit/Pkg	Pkg/Case
19910544	Shoe Cover Non-Conductive Non-Skid-XL - Blue	100	3
19910545	Shoe Cover Non-Conductive Non-Skid-Reg - Blue	100	3
19910546	Shoe Cover Non-Conductive Reg - Blue	100	3
19910802	Shoe Cover Non-Conductive Non-Skid-XL - White	100	3

Distributed by Fisher Scientific. Contact us today:

### In the United States

Order online: [fishersci.com](http://fishersci.com)  
Call customer service: 1-800-766-7000

### In Canada

Order online: [fishersci.ca](http://fishersci.ca)  
Call customer service: 1-800-234-7437